

FirstTemp*
Genius*

Infrared Thermometry



Everyone benefits from the touch of GENIUS

tyco

Healthcare

For accuracy and patient comfort

Trust the touch of GE

How does the GENIUS thermometer work?


The tympanic membrane shares the same blood supply with the hypothalamus, the controller of body temperature. The *FirstTemp* GENIUS infrared thermometer has been specifically designed to provide one of the most accurate non-invasive measurements of core body

temperature by processing and displaying infrared readings from the tympanic membrane.


With the GENIUS thermometer in the tympanic mode and with a new probe cover in place, the probe tip is gently placed into the external auditory canal until a seal is made.

On depress
under 2 s
32 individ
GENIUS "P
possible is c

Infrared thermometry

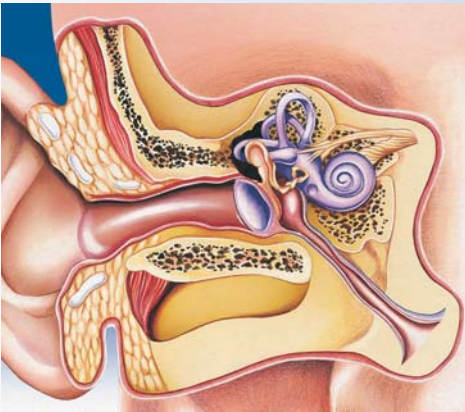

GENIUS unique "Peak Select" system

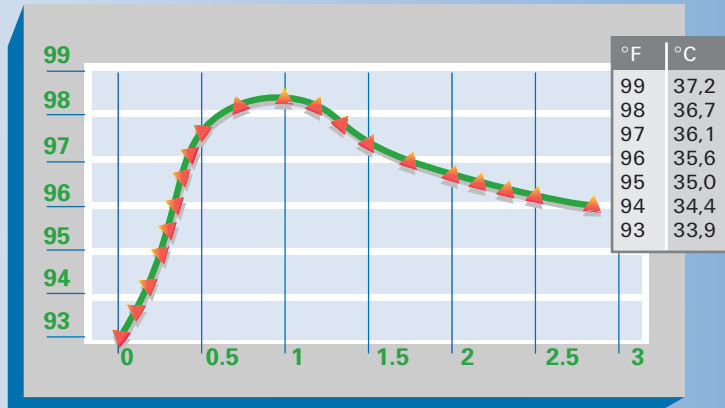

Surface



Get more from your GENIUS thermometer

The *FirstTemp* GENIUS infrared thermometer has not only been designed to provide quick and accurate information:

- It allows the choice of display in oral, rectal or core equivalent values.
- A mode selection button allows the choice of tympanic assessment, surface mode, pulse timer and a °C/°F option.
- A clear prompt window displays both accurate temperature readings in under 2 seconds and simple step by step instructions.

- A release button ejects disposable probe covers into a waste receptacle for safe, hygienic operation.
- The base is impact resistant and holds 21 no-touch disposable probe covers.
- The GENIUS thermometer is lightweight and portable for use by all healthcare professionals.

Neither the patient nor the healthcare professional suffers discomfort, embarrassment or distress. GENIUS thermometers can provide versatility and accuracy to all healthcare workers.

GENIUS

ing the scan button, the temperature is taken in seconds. The GENIUS thermometer takes multiple infrared readings and using the unique Peak Select™ system, the most accurate reading is displayed.

scanning mode



INFRARED THERMOMETRY

ACCURATE

NON-INVASIVE

NO CROSS INFECTION

SPEED & EASE OF USE

VERSATILITY

LATEST TECHNOLOGY

SAFE

Commitment to Customer Care

FirstTemp GENIUS thermometers provide more than just fast, accurate and reliable temperature readings. As befits a product at the forefront of its kind, the *FirstTemp* GENIUS thermometer comes with a unique 3 year warranty and repair service.

Furthermore the *FirstTemp* GENIUS thermometer is supported by a comprehensive in-service and educational back-up programme.

Trained company representatives will hold in-hospital training sessions to familiarise all users with GENIUS thermometers as well as provide valuable material by way of booklets, posters and training videos. Training seminars and lectures are also available for large groups.

The company also has a service centre in the UK that offers technical support to customers. Having this ISO 9001 approved facility on hand allows for the highest quality, and possibly, the most rapid technical support service available.

Mode Selection Button
Allows you to choose tympanic assessment, surface mode or pulse timer. Also used to change display temperature from °C to °F.

Scan Button
Activates GENIUS thermometer for instantaneous temperature measurement.

Release Button
Ejects disposable probe cover into waste receptacle for safe, hygienic operation.



Ordering Information

Catalogue Number	Description	Quantity per Carton	NSV Code
8884100010	GENIUS thermometer	1	
8884110019	GENIUS probe covers	1260 (12 x 105)	FWH025
8884814065	GENIUS wall mount	1	
8884830285	GENIUS model 3000PC calibrator	1	
7774830140	GENIUS desktop interface kit	1	
7774830215	GENIUS laptop interface kit	1	



* Trademark of Sherwood Services AG. © 2004 Tyco Healthcare Group LP. All rights reserved.

P-TH-FTGenius01/GB

tyco
Healthcare

Tyco Healthcare Deutschland GmbH
Josef-Dietzgen-Straße 1
D-53773 HENNEF
Tel.: +49 (0) 2242 887275
Fax: +49 (0) 2242 887199

Tyco Healthcare Norden AB
Gårdsvägen 18
S-16967 SOLNA
Tel: +46 (0) 8585 60 505
Fax: +46 (0) 8585 60 506

Tyco Healthcare Spain S.L.
Fructuos Gelabert, 6 8ª planta
08970 Sant Joan Despí
BARCELONA
Tel: +34 93 475 86 68
Fax: +34 93 373 87 10

Tyco Healthcare Italia SpA
Via Rivoltana, 2/d
I-20090 SEGRATE (Mi)
Tel: +39 02 70317.1
Fax: +39 02 7530125

Tyco Healthcare Nederland BV
Hogeweg 105
NL-5301 LL ZALTBOMMEL
Tel: +31 (0) 418 57 66 00
Fax: +31 (0) 418 57 67 93

Tyco Healthcare Belgium N.V.-S.A.
Generaal de Wittelaan 9/5
B-2800 MECHELEN
Tel: +32 (0) 15 29 44 50
Fax: +32 (0) 15 29 44 55

Tyco Healthcare Austria GmbH
Campus 21, Europaring F09402
A-2345 Brunn am Gebirge
Tel: +43 (0) 2236 37 88 39
Fax: +43 (0) 2236 37 88 39 40

Tyco Healthcare Switzerland Ltd.
Roosstrasse 53, Postfach
CH-8832 WOLLERAU
Tel: +41 1 786 50 50
Fax: +41 1 786 50 10

Tyco Healthcare France S.A.
3, avenue du Canada - Les Ulis
F-91975 COURTABOEUF-CEDEX
Tel: +33 (0) 1 69 82 15 18
Fax: +33 (0) 1 69 82 15 28

Tyco Healthcare Regional Export
154 Fareham Road
GOSPORT
HAMPSHIRE
UK - PO13 0AS
Tel: +44 (0) 13 29 22 42 58
Fax: +44 (0) 13 29 22 40 86

Tyco Healthcare Portugal
Produtos de Saúde Lda.
Est: Outeiro Polima, Lote 10-1º Piso
Abóboda
P-2785-521 S. Domingos de Rana
Tel: +351 21 448 10 00
Fax: +351 21 445 05 88

Tyco Healthcare South Africa
Birchwood Court
Montrose Road, P.O. Box 8108
Halfway House
1685 MIDRAND
SOUTH AFRICA
Tel: +27 (0)11 265 2000
Fax: +27 (0)11 265 2050

Tyco Healthcare UK Limited
154 Fareham Road
GOSPORT
HAMPSHIRE
UK - PO13 0AS
Tel: +44 (0) 13 29 22 41 59
Fax: +44 (0) 13 29 22 40 83
www.tycohealthcare.co.uk

TY/0035